SONDE SAINT-CLOUD

CefarCompex 

- **SONDE ANALE DE REEDUCATION :
STIMULATION ET BIOFEEDBACK**
- **PROBE FOR ANAL STIMULATION:
STIMULATION AND BIOFEEDBACK**
- **REHABILITATIONS-ANAL:
STIMULATION UND BIOFEEDBACK**
- **SONDA ANAL DE REEDUCACIÓN:
ESTIMULACIÓN Y BIOFEEDBACK**
- **ANALSONDE TER REVALIDATIE:
STIMULERING EN BIOFEEDBACK**
- **SONDA ANALE DI RIEDUCAZIONE:
STIMULAZIONE E BIOFEEDBACK**
- **ÄNDTARMSSOND FÖR REHABILITERING:
STIMULERING OCH BIOFEEDBACK**

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WARNING

This probe is strictly for personal use.

CLEANING AND MAINTENANCE

- The SAINT-CLOUD anal probe should be cleaned and/or disinfected (using cold disinfectant) and then rinsed BEFORE USING IT FOR THE FIRST TIME and AFTER EVERY USE.
- DO NOT DIP CONNECTOR IN WATER

EQUIPMENT RECOMMENDATIONS

This probe should only be used with equipment conforming to Directive 93/42/CEE with the following characteristics:

- Two phase, zero net AC
- Maximum intensity: 150 mA
- Maximum output voltage: +/- 80 V

NOT RECOMMENDED FOR USE IF:

- Wearing a pacemaker(*)
- STD
- Suffering from total denervation
- Suffering from cancer in the rectal area
- Suffering from rectal infection
- Haemorrhoid
- Pregnant

(*) In case of doubt seek advice from your doctor or medical specialist

INSTRUCTIONS FOR USE

CAUTION : Verify the integrity of the probe before each use. In case of damage, do not use the product.

METHOD OF USE

- 1 - Wash the probe before using it for the first time (see « Warning »).
- 2 - Lubricate the probe with lubricating gel to ease insertion (using a gel designed for this purpose).
- 3 - Insert the probe (disconnected from the equipment) in the anal orifice until the end part of the probes penetrate the rectal cavity and that the sphincter close around the probe.
- 4 - Connect the two cables to your equipment adapter.
- 5 - Start the stimulator and gradually increase the intensity of the stimulation.
- 6 - At the end of the session switch off the equipment and disconnect the probe.
- 7 - Remove the probe from the rectal area and clean or disinfect it as indicated above under "Warning".
- 8 - The probe should be stored in its original packaging, which should be kept throughout the period of treatment (the serial number is shown on the packaging).

UTILITY OF THE STRIP AND THE COLLAR

The upholding of the probe is ensure by the use of the strip that will avoid the expulsion from the anal cavity. This strip is inclinable for comfort of use

The angle between the collar and the strip can go from 70° and 90° and enable a better positioning of the probe.

RARE SIDE EFFECTS

- Irritation

ATTENTION

- The Patient should only be connected to a single stimulator. Do not connect the probe simultaneously to high frequency surgical equipment.
- Maximum setting 150 mA (voltage +/- 80V from peak to peak at 500Ω).
- Never insert or remove the probe while stimulator is operating.

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